Desgoquat

DE	Aldehydfreies flüssiges Desinfektionsreiniger-Konzentrat	2
EN	Liquid aldehyde-free disinfectant cleaner concentrate	4
FR	Nettoyant désinfectant concentré liquide exempt d'aldéhydes	6
IT	Disinfettante liquido concentrato per superfici. Battericida, fungicida e virucida	8
NL	Aldehydevrije vloeibare desinfectiereiniger-concentraat	10
HU	Folyékony, aldehydmentes tisztító- és fertőtlenítő koncentrátum	12
RO	Detergent dezinfectant lichid concentrat fără aldehyde	14
DK	Koncentreret flydende desinfektions- og rengøringsmiddel uden aldehyder	16
NO	Konsentrert aldehydfri desinfeksjons- og rengjøringsmiddel	18
RU	Безальдегидный жидкий дезинфицирующий чистящий концентрат	20
PL	Płyn do mycia i dezynfekcji w koncentracji, wolny od aldehydów	22
CZ	Tekutý dezinfekční čistící prostředek - koncentrát bez aldehydů	24
SK	Aldehydy neobsahujúci, tekutý čistiaci a dezinfekčný koncentrát	26
EE	Aldehydühivaba desinfitseeriv puhastusaine kontsentraat	28
LT	Skystas koncentruotas dezinfekcinis valiklis be aldehydų	30
LV	Aldehydus nesaturoš šķidrš dezinfekcijas līdzeklis, koncentrāts	32
HR	Tekuće koncentrirano sredstvo bez aldehida za čišćenje i dezinfekciju površina	34
RS	Dezinfekciono sredstvo za čišćenje bez aldehida, koncentrat	36
PT	Produto de limpeza desinfectante líquido concentrado sem aldeídos	38
GR	Καθαρό συμπύκνωμα απολυμαντικού χωρίς υγρά αλδεϋδη	40

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Liquid aldehyde-free disinfectant cleaner concentrate

Application

For the cleaning and disinfection of water and alkali-resistant floor, wall, ceiling and furnishing surfaces in hygiene-relevant sectors.

Directions

Please test on a small inconspicuous spot before first application. Kiehl accepts no liability or responsibility for damage caused as a result of incorrect use or application of the product.

Preparing of the ready-to-use solution:

1 dosage = 25 ml
Always prepare solution with cold water. Dilution: see table.
Suitable dosing systems: DosiStation, DosiJet, Arcantec.

Disinfection and cleaning:

Fully cover surface with disinfection solution (approx. 15–20 ml/m²) and clean with a cleaning cloth or cleaning device. Surfaces and kitchen equipment that come into direct contact with food must be rinsed thoroughly with drinking water once the product has taken effect.

Necessary risk minimisation measures

Remove spilled concentrate immediately, rinse thoroughly with clear water. Wear personal protective equipment.

First Aid

General information: Avoid contact with skin and eyes.

After inhalation: Move to fresh air. If unconscious place in recovery position and seek medical advice. If breathing is irregular or stopped, administer artificial respiration. No artificial respiration, mouth-to-mouth or mouth to nose. Use suitable instruments/apparatus. Give oxygen. First aider needs to protect himself. Call a physician or poison control centre immediately. Keep respiratory tract clear. Call a physician immediately.

After contact with skin: After contact with skin, wash immediately with plenty of water and soap. Take off contaminated clothing and shoes immediately. Immediate medical treatment required because corrosive injuries that are not treated are hard to cure. Take victim immediately to hospital.

After contact with eyes: Rinse immediately with plenty of water, also under the eyelids, for at least 15 minutes. Call a physician immediately. Remove contact lenses. Protect unharmed eye. Keep eye wide open while rinsing. Continue rinsing eyes during transport to hospital. Small amounts splashed into eyes can cause irreversible tissue damage and blindness.

After ingestion: Call a physician immediately. Clean mouth with water and drink afterwards plenty of water. Do not induce vomiting without medical advice. Never give anything by mouth to an unconscious person. Take victim immediately to hospital.

Storage

Requirements for storage rooms and vessels: Keep container tightly closed in a cool, well-ventilated place. To maintain product quality, do not store in heat or direct sunlight. Containers which are opened must be carefully resealed and kept upright to prevent leakage. Electrical installations / working materials must comply with the technological safety standards. Store in a place accessible by authorized persons only.
Hints on joint storage: Keep away from food, drink and animal feedings. Do not store near acids.

Further information on storage conditions: Never return unused material to storage receptacle. Protect from frost.

Disposal

Disposal recommendations: Do not dispose of waste into sewer.

List of Wastes Code - residues/unused products: 070699 WASTES FROM ORGANIC CHEMICAL PROCESSES; wastes from the MFSU of fats, grease, soaps, detergents, disinfectants and cosmetics; wastes not otherwise specified.

List of Wastes Code - used product: 070699 WASTES FROM ORGANIC CHEMICAL PROCESSES; wastes from the MFSU of fats, grease, soaps, detergents, disinfectants and cosmetics; wastes not otherwise specified.

Contaminated packaging: Clean container with water. Return cleaned containers to the company for recycling. Offer rinsed packaging material to local recycling facilities.

Cleaning of equipment

After use, clean all parts of the equipment thoroughly with water, disinfect and dry completely before using it again.

Recommended dosage for surface disinfection in industrial and institutional areas and food industry

Test method	Spectrum of activity	Organic load	Dilution	Contact time	Test microorganisms
EN 13697 Surface disinfection without mechanical action	bactericidal yeastcidal	high	2.5% 250 ml/10 L	15 min	Staphylococcus aureus incl. MRSA, Enterococcus hirae, Escherichia coli, Pseudomonas aeruginosa, Candida albicans
EN 14476 Quantitative suspension test	virucidal against enveloped viruses	high			Modified Vacciniavirus Ankara (MVA)

Recommended dosage for surface disinfection in the healthcare sector

EN 16615 / 4-field test VAH-certified application recommendation for surface disinfection with mechanical action	bactericidal yeastcidal	high	2% 1% 200 ml/10 L 100 ml/10 L	15 min 30 min	Staphylococcus aureus incl. MRSA, Enterococcus hirae, Pseudomonas aeruginosa, Candida albicans
EN 14476 Quantitative suspension test	virucidal against enveloped viruses	high			Modified Vacciniavirus Ankara (MVA)
EN 13697 Surface disinfection without mechanical action	bactericidal yeastcidal	high	1% 100 ml/10 L	60 min	Staphylococcus aureus incl. MRSA, Enterococcus hirae, Pseudomonas aeruginosa, Candida albicans
EN 14476 Quantitative suspension test	virucidal against enveloped viruses	high			Modified Vacciniavirus Ankara (MVA)

Test method	Spectrum of activity	Organic load	Dilution	Contact time	Test microorganisms	
Further tests						
Medical area						
EN 16615 / 4-field test VAH-certified application recommendation for surface disinfection with mechanical action	bactericidal yeastcidal	high	2% 1% 200 ml/10 L 100 ml/10 L	1 min 30 min	Staphylococcus aureus incl. MRSA, Enterococcus hirae, Pseudomonas aeruginosa, Candida albicans	
EN 13697 Surface disinfection without mechanical action	bactericidal yeastcidal	high	7% 1% 300 ml/10 L 100 ml/10 L	5 min 60 min	Staphylococcus aureus incl. MRSA, Enterococcus hirae, Pseudomonas aeruginosa Candida albicans	
EN 13727 Quantitative suspension test	bactericidal	high	2% 1% 200 ml/10 L 100 ml/10 L	1 min 60 min	Staphylococcus aureus incl. MRSA, Enterococcus hirae, Pseudomonas aeruginosa	
EN 13624 Quantitative suspension test	yeastcidal	high	4% 1% 0.25% 400 ml/10 L 100 ml/10 L 25 ml/10 L	1 min 5 min 60 min	Candida albicans	
Industrial and institutional areas, food industry						
EN 1276 Quantitative suspension test	bactericidal	high	2% 1.5% 200 ml/10 L 150 ml/10 L	1 min 5 min	Staphylococcus aureus incl. MRSA, Enterococcus hirae, Escherichia coli, Pseudomonas aeruginosa, Listeria monocytogenes, Salmonella typhimurium	
EN 1650 Quantitative suspension test	yeastcidal	high	1% 0.5% 100 ml/10 L 50 ml/10 L	1 min 15 min	Candida albicans	
EN 13697 Surface disinfection without mechanical action	bactericidal yeastcidal	high	2% 200 ml/10 L	15 min	Staphylococcus aureus incl. MRSA, Enterococcus hirae, Escherichia coli, Pseudomonas aeruginosa, Candida albicans	
	Listeria	high	1.5% 150 ml/10 L	5 min	Listeria monocytogenes	
	salmonella	high	3% 300 ml/10 L	5 min	Salmonella Typhimurium	
Virucidal activity						
EN 14476 Quantitative suspension test	virucidal	low	4% 5% 4% 5% 2% 4% 2% 2% 2% 2% 4% 2% 4% 2% 4% 2% 4% 2% 4% 2%	400 ml/10 L 500 ml/10 L 400 ml/10 L 500 ml/10 L 200 ml/10 L 400 ml/10 L 200 ml/10 L 200 ml/10 L 200 ml/10 L 200 ml/10 L 400 ml/10 L 200 ml/10 L 400 ml/10 L 200 ml/10 L 400 ml/10 L 200 ml/10 L 400 ml/10 L 200 ml/10 L 400 ml/10 L 200 ml/10 L	30 min 60 min 30 min 60 min 15 min 15 min 60 min 5 min 15 min 15 min 15 min 60 min 15 min 60 min 5 min 5 min 1 min 5 min 60 min	Poliovirus, Adenovirus, Murine Norovirus (MNV) Poliovirus Adenovirus Murine Norovirus (MNV) enveloped viruses, Adenovirus, Murine Norovirus (MNV) Modified Vacciniavirus Ankara (MVA) Influenza-A Virus H7N9, Influenza-A Virus H1N1 BCoV (surrogate for Coronavirus incl. MERS-CoV) Duck-HBV (surrogate for HBV) Adenovirus
EN 16777 Quantitative surface test	Adenoviruses	high	5% 500 ml/10 L	60 min	Adenovirus	
DVV/RKI Quantitative suspension test	virucidal	with	5% 4% 5% 4% 4% 2% 2% 5% 3% 3%	500 ml/10 L 400 ml/10 L 500 ml/10 L 400 ml/10 L 400 ml/10 L 200 ml/10 L 200 ml/10 L 500 ml/10 L 300 ml/10 L 300 ml/10 L	15 min 60 min 15 min 60 min 30 min 30 min 5 min 30 min 60 min 15 min	Poliovirus, Adenovirus, Polyomavirus, Vacciniavirus Poliovirus Adenovirus Polyomavirus Vacciniavirus Bovine Enterovirus (ECBO-Virus) Rotavirus

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KIEHL Italia S.r.l.
KIEHL Schweiz AG
KIEHL Middle East LLC